

Mounting the ADEP grips

Preparation

1. Container
2. Petroleum ether / spirit (available in pharmacies and drugstores)
3. Tweezer
4. Cloth
5. Wire insulating
6. Instrument
7. ADEP grips



Mounting the ADEP grips

1. Dip the ADEP grip into the petroleum ether for 20 to 45 seconds. Attention: no more than 45 seconds for an optimal result.
2. Take the ADEP grip with the tweezer, drain it a little and wipe it dry on the cloth.
3. Slip a piece of wire insulating over the tip of the instrument to protect the ADEP grip.
4. Hold the ADEP grip like on the picture below and slide it along the instruments curve. Press a little with your ring finger on the back of the instrument in order to better follow the curve.



Repeat the procedure with the second grip.



Pay attention to place the sleeves correctly. Control it with the pictures below.



By following exactly these instructions, your ADEP grips will fit perfectly for the whole instrument's life. The grips are not meant to be re-used on other instruments.